

IDEAL SOLUTION FOR FEEDING NUTRITION

Free your hands and worry



COMMON HEADACHES DURING FEEDING


WHAT CAN I SOLVE




FEEDING INTERMITTENTLY



UNSTABLE IN FEEDING SPEED

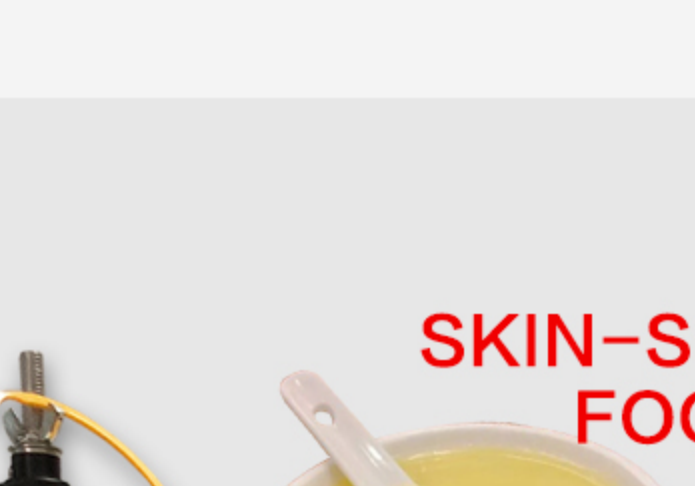

EXHAUSTED CAREGIVER

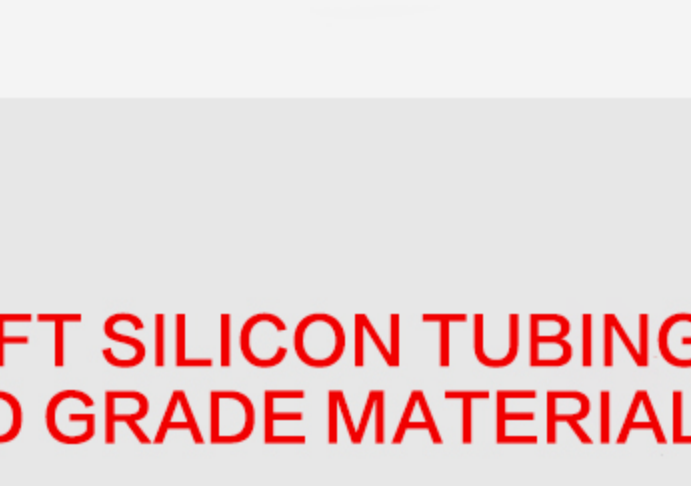

LONG PROCESS OF CLEANING

SKIN-SOFT SILICON TUBING FOOD GRADE MATERIAL

Safe & Stable feeding process no spill or splash



One button design easy to operate


ONE BUTTON TUBING CLEANING

Easier and convenient




FEATURES OF THIS PRODUCT

1. No matter the nutrition is thin like milk, solution, juice or thick like tomato sauce, home-made liquid food, this machine could do the work.
2. With adjusted speed, the patient could have food per his/her own preference.
3. One button design with minimum training needed to operate it.
4. Compatible with multiple feeding method, nasal, Gastric tube, IOE, Gastrostomy, Intestines tube, ect.
5. Reusable Tubing, suitable for water boiling disinfection
6. More than 6000hours lifetime guaranteed.

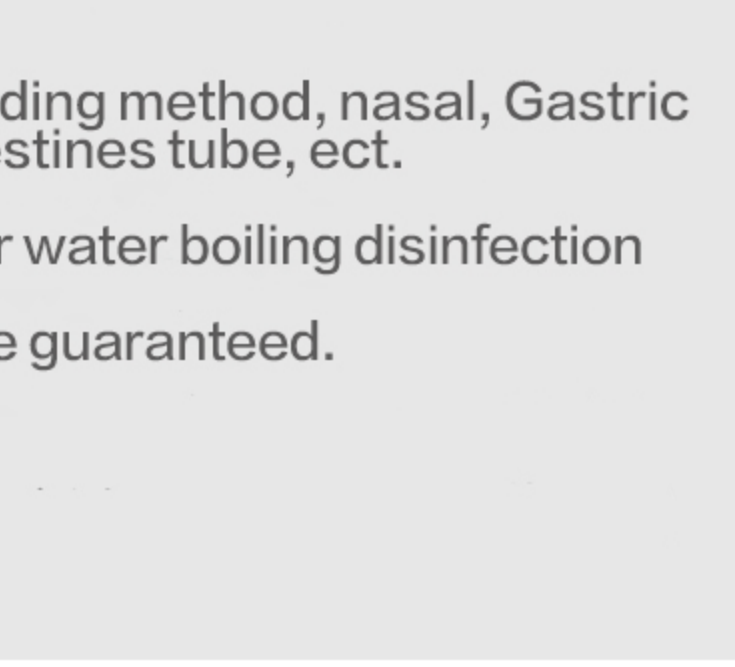

PRODUCT PARAMETERS

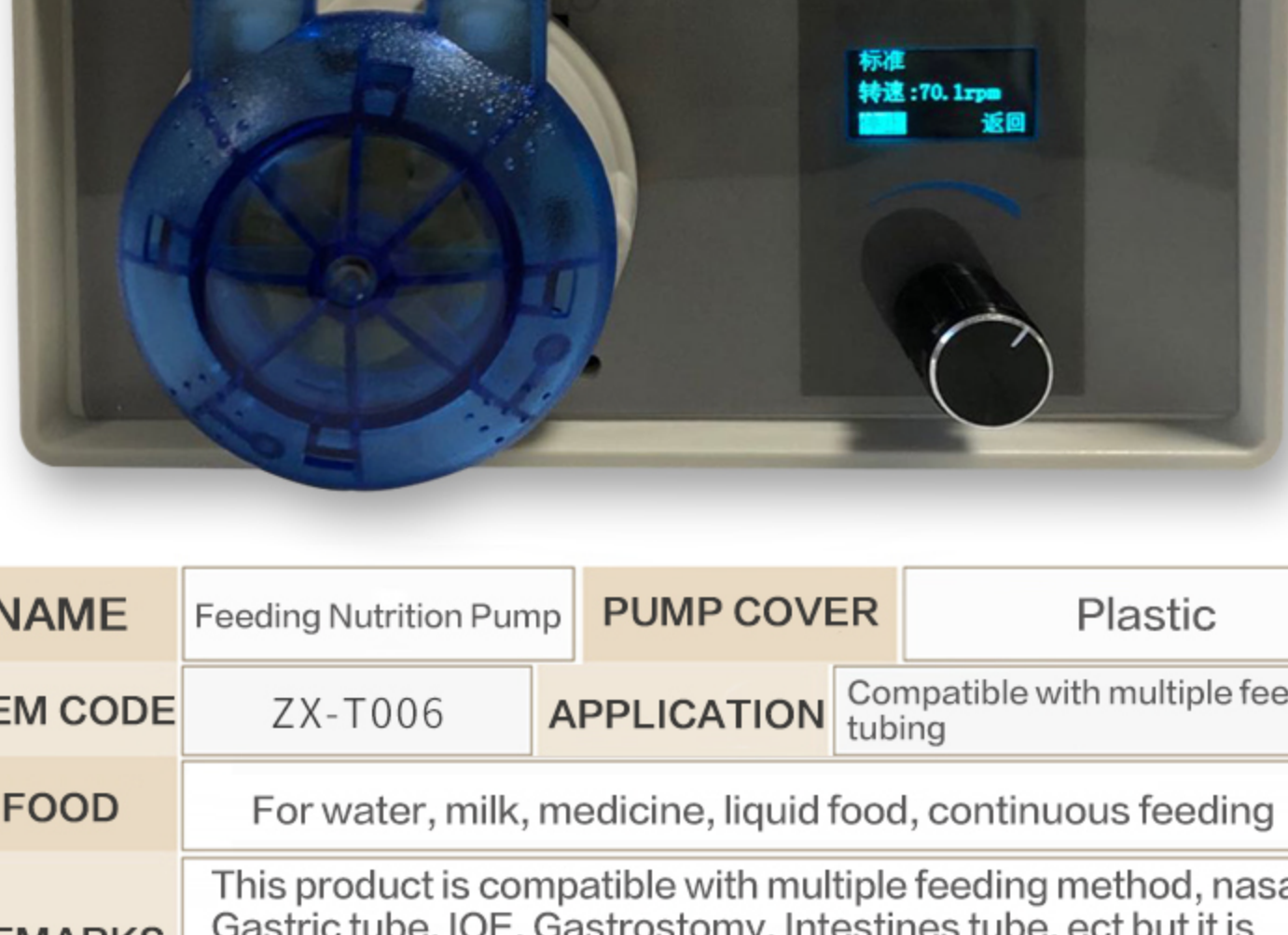

NAME	Feeding Nutrition Pump	PUMP COVER	Plastic
ITEM CODE	ZX-T006	APPLICATION	Compatible with multiple feeding tubing
FOOD	For water, milk, medicine, liquid food, continuous feeding		
REMARKS	This product is compatible with multiple feeding method, nasal, Gastric tube, IOE, Gastrostomy, Intestines tube, ect but it is not suggested to operate alone and will need someone to look after the whole process.		


SUITABLE NUTRITION

It suits for both thin and thick




OPERATION PROCEDURE

1. Got the nutrition ready.
2. Make sure the tubing is clean.
3. Connect the tubing with machine.
4. Prime the sets with water.
5. Connecting the nutrition then start feeding in proper speed.
6. after each feeding completed, cycle flush the tubing with warm water .
7. Check to confirm tubing is clean.
8. Tidy up the tubing and machine for next time use.

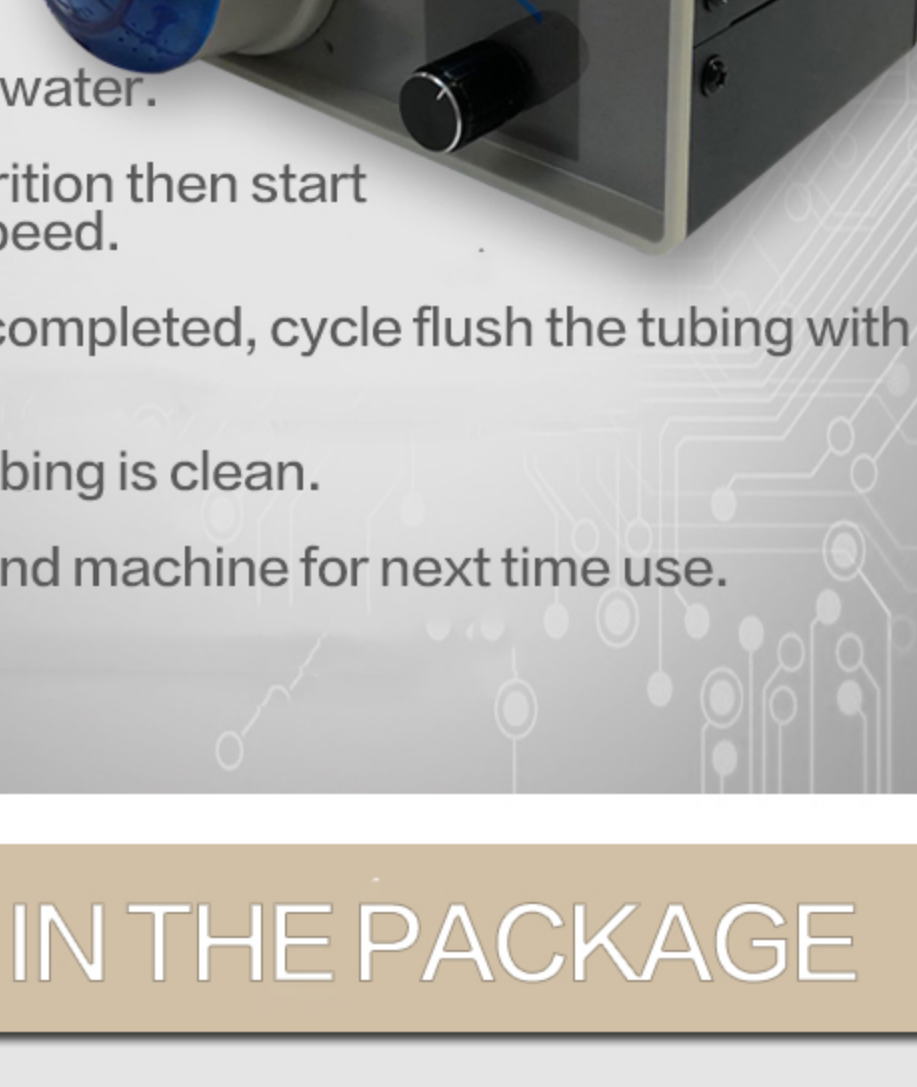

WHAT IS IN THE PACKAGE

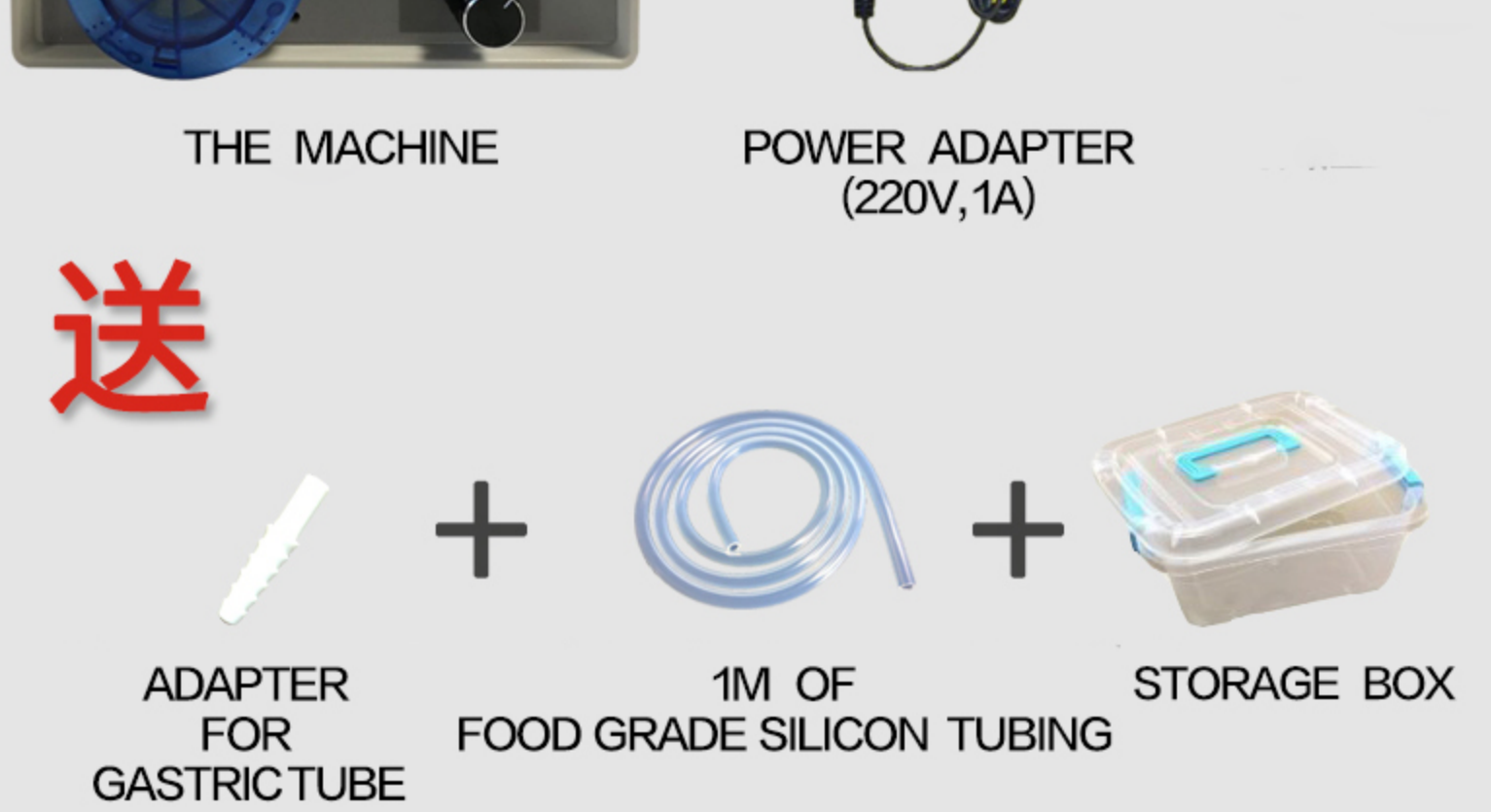

THE MACHINE


POWER ADAPTER (220V,1A)

ADAPTER FOR GASTRICTUBE

1M OF FOOD GRADE SILICON TUBING

STORAGE BOX

SUITABLE FOR

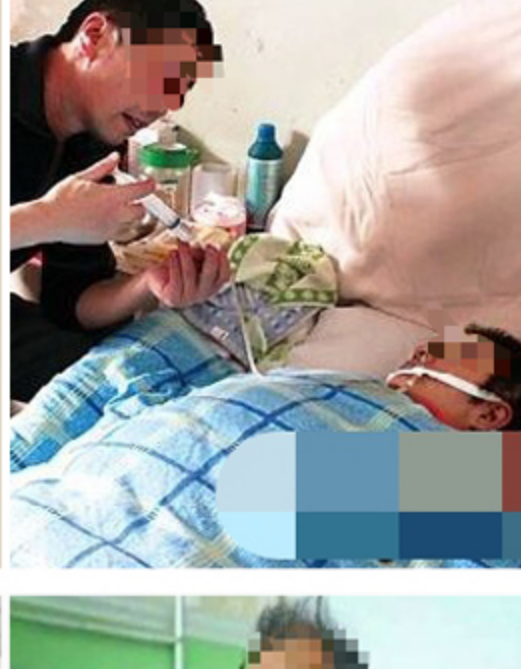

Person Difficulty swallowing, Gastrostomy, Digestion abnormality, Nasal feeding,
But the patient should not operate this machine alone and will need someone to look after the whole process.